


For lifelike esthetics and precise shading

IPS e.max ZirCAD Prime and IPS e.max ZirCAD Prime esthetic can offer layer-free restorations, ensuring an unique seamless progression of shade, optimal translucency levels and precise shading. This provides the ideal basis for premium esthetics in monolithic, fully and partially veneered restorations.


Wall-thickness Guideline for reliable processes


In preparing teeth for restorations made of IPS e.max ZirCAD Prime and IPS e.max ZirCAD Prime esthetic, it is important to observe the general guidelines for all-ceramics, for example, rounded angles and edges, in addition to the required minimum wall-thickness (specifications in mm for monolithic restorations).

 Anterior crowns

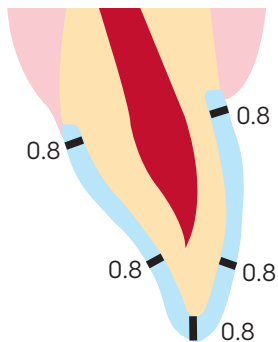

 Anterior bridges
(abutments)

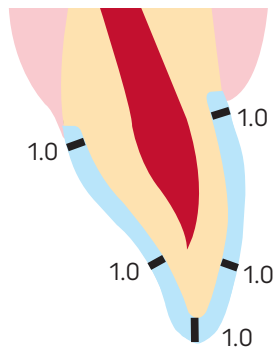

 Posterior crowns
and bridges

